ADVOCATE®

Redi-Code⁺ Speaking

Quick Reference Guide

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1 Start with the meter off. Press Set button (the back of the meter). After setting the Year, Month, Day, Hour, Minutes and Alarms you must set your Speaking Language. Default language is English, press M button to set it to Spanish. Now set speaking volume.

Press M button to adjust the speaking volume (0 for the silent mode and 6 for the largest speaking volume). Press Set button to move to memory deletion mode.

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2 INSERT A TEST STRIP TO TURN ON THE METER.

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Dear ADVOCATE REDI-CODE Plus System Owner:

Thank you for choosing ADVOCATE Redi-CodePlus Blood Glucose Monitoring System to take control of your diabetes. This system is helpful in tracking your blood glucose values. Your Quick Reference Guide assists you to set up the system and helps you to run the test. But if you'd like more information and additional features, please check the User Manual. Please read this Guide carefully before use, or you can read the User Manual for complete information.

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3 CHOOSE TEST MODES. When is flashing press M button to choose test modes (pre-meal/post-meal/general)

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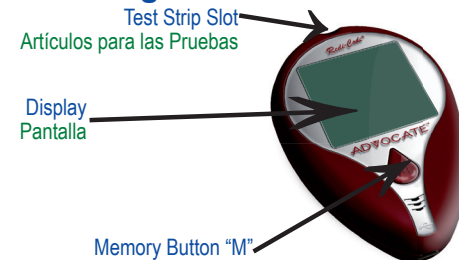
4 While is flashing on the display, apply blood to the absorbent hole of the test strip until the confirmation window is completely covered.

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The blood should be completely filled in the confirmation window. If you find that the confirmation window is not filled with blood when the meter is counting, NEVER try to add more blood to the test strip. Discard the test strip and retest with a new one.

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5 After the meter counts to 5, your test result appears along with Date/Time/Result indicator and is automatically stored in the memory.

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Diabetic Supply of Suncoast, Inc.